

**Instructions for installing the device and accessories on
the cart, for the following devices:**

**AFERSMART
LIPIDSMART
VITASMART
CARDIOSMART
LEUKOSMART**

VitaSmart
Duo

CE
0476

Produced by:

MEDICA

MEDICA S.p.A.

Via Degli Artigiani, 7
41036 MEDOLLA
MODENA – ITALY

Tel. +39 0535 51159 Fax+39 0535 52605

Website www.medica.it E-mail: info@medica.it




Device installation

CONTENTS

1	Assembly of the machine body on trolley	3
2	Assembly of the power strip on the trolley	4
3	Assembly of the console	6

Device installation




1 ASSEMBLY OF THE MACHINE BODY ON TROLLEY

1	Position the machine on the trolley and secure it with the four TCEI M6x10 screws supplied	
2		
3	Connect the trolley ground cable to the bonding connection terminal of the machine	

Device installation

2 ASSEMBLY OF THE POWER STRIP ON THE TROLLEY

The power strip, code **T14530**, is an OPTIONAL accessory of the device.

1	Remove the screws on the rear side of the trolley.	
2	Fasten the power strip to the two holes on the trolley with the two TCEI 4 x 16 screws and flat washer supplied.	
3	Position it so that the labeling is at the bottom.	

Device installation



- 4 Image of trolley with power strip installed



Device installation

3 ASSEMBLY OF THE CONSOLE

The Console, Code M90281, is an optional accessory of the device

1	<p>First, unscrew the bag support from the trolley stand to insert the console support</p>	
2	<p>Secure the white plastic component Code M7540206 on the trolley stand with 4 TCEI 3x14 screws (supplied). The component must be positioned at the height at which you want to mount the console.</p>	

Device installation

3


Insert second component Cod. T34254 on that just set and tighten the top as in picture.



Device installation

4

Once the arm is set you can set the console: the first thing to do is remove the plastic cap (see photo).

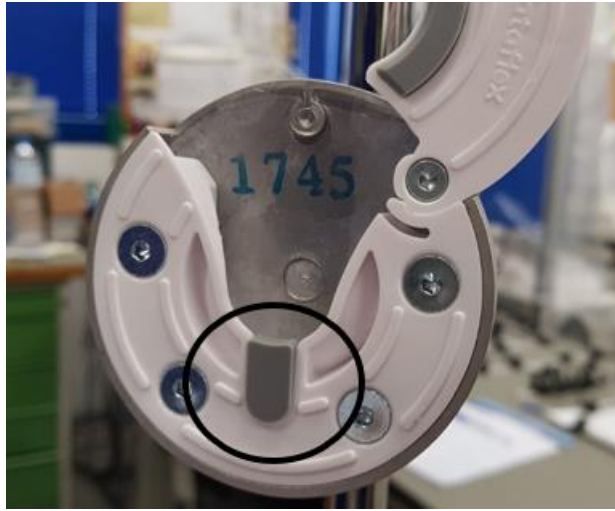




5

Insert the console Code M7540185 in its place.



Device installation

		
6	<p>Close the locking lever to fully secure the console to the arm.</p>	
7	<p>Connect the console to the body using the supplied serial cable.</p>	

Device installation

8

Connect the console power cord to the power strip on the back of the trolley.



9

To open the safety lock of the power strip, press the point indicated in the picture with a pointed tool.

WARNING: Close the safety lock once the power cord has been inserted

