



**Disposable Set for Portal Vein
Perfusion of the Liver**

KOP03L

Instructions for Use

Bridge to Life (Europe) Ltd.

Tel: +44(0)20 3411 8326 | Mobile: +44(0)77 6652 7191 | www.bridgetolife.eu

LU 311 The Light Bulb | 1 Filament Walk | London SW18 4GQ



DISPOSABLE SET FOR PORTAL VEIN PERFUSION OF THE LIVER KOP03L

Via Degli Artigiani, 7 - 41036 Medolla (Modena) - ITALY

Tel. +39 0535 51159 - Fax +39 0535 52605


info@medica.it - www.medica.it

The disposable set is composed of the following items:

Q.ty	Ref	Description
	KOP03L	Kit per la perfusione della Vena porta del fegato da trapiantare / Kit for organ perfusion in liver transplantation (portal vein)
1	M90356	Linea unica perfusione vena fegato - ETO STERILE / Single line for liver perfusion (portal vein) - ETO STERILE
1	M27402	IFU perfusione organo fegato (vena porta) / IFU liver perfusion (portal vein)
1	EU4391	TUBO PER OSSIGENO CON FILTRO GAS - ETO STERILE / OXY TUBE WITH GAS FILTER - ETO STERILE
1	EU0924	IFU per tubo ossigeno EUROSETS / IFU OXY TUBE
1	M90362	Accessorio tag per perfusione fegato (vena porta) - NS / Tag accessory for liver perfusion (portal vein) - NS

Instructions

- ✓ Extract the passive RFID tag, ref M90362 placed inside a pocket fixed to the inner side of the box lid. The tag contains information about the Disposable set reference, lot and expire date
- ✓ Insert the RFID tag into the reader on the front panel of the Vitasmart. Keep it on the reader for the whole duration of the procedure. If the RFID tag matches the selected procedure, the Vitasmart will accept it and goes to the next step. If not the machines stops until a new/proper tag is inserted.
- ✓ Extract the line for Liver perfusion ref M90356 from the box

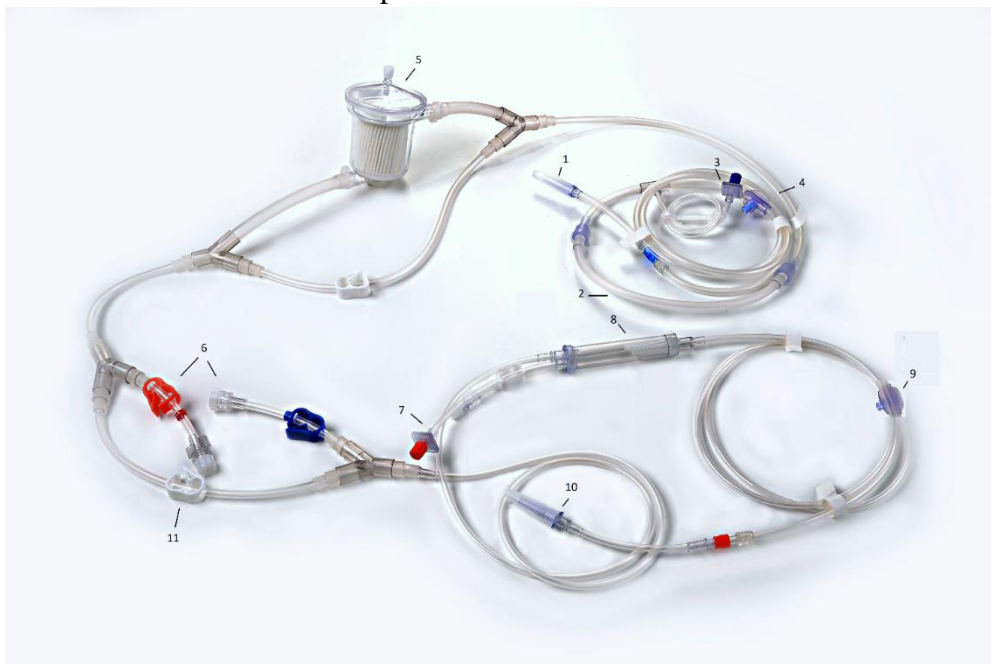

DISPOSABLE SET FOR PORTAL VEIN PERFUSION OF THE LIVER

KOP03L

Via Degli Artigiani, 7 - 41036 Medolla (Modena) - ITALY

Tel. +39 0535 51159 - Fax +39 0535 52605

info@medica.it - www.medica.it

- ✓ Put the pump segment (2) on the rollers of the anterior pump of Vitasmart. Refers to the user manual of Vitasmart, too.
- ✓ Connect the line with the Blue Transducer Protector (3) to the Blue pressure connector on the front panel of Vitasmart
- ✓ Put the filter (5) on the specific holder
- ✓ Insert the chamber (8) into the black holder on the front panel of Vitasmart
- ✓ Insert the line exiting the chamber in the Vitasmart air bubble detector
- ✓ Connect the line with the Red Transducer Protector (7) to the Red pressure connector on the front panel of Vitasmart

When using the Oxygenator

- ✓ Insert the Oxygenator ref EU5054 into the holder of the Vitasmart
- ✓ Connect the line segments (6) to the respective connectors of the Oxygenator connectors making attention that the colors match. Refer also to the Oxygenator EU5054 user manual.
- ✓ Connect the tubing ref EU4391 to the oxygen inlet of the Oxygenator. Plug the other end of the tubing to the oxygen blender of the oxygen outlet.
- ✓ Close the white clamp (11)

When not using the Oxygenator

- ✓ Close the clamps on the line segments (6)
- ✓ Open the white clamp (11)

In both cases

- ✓ Remove the cap and immerse the line end (1) in the bowl containing the perfusion solution.
- ✓ Remove the cap and put the line end (10) in the bowl.
- ✓ Begin the priming procedure on the Vitasmart (refer to its manual)
ATTENTION: During the priming, the organ must NOT be connected to the line.
- ✓ After 200 ml of solution have flown trough the line, close the clamp on the inferior bypass segment of the filter (5).
- ✓ Fill the chamber (8) up to 2/3rd. To do so remove the cap on the top pf the chamber. Replace the cap when the chamber is filled.
- ✓ Verify that solution flows through the line and comes out from the line end (10)
- ✓ When the circulated volume of solution reaches 600 ml, it is possible to stop the priming and starting the organ perfusion.
- ✓ Connect the line end (10) to the cannula previously inserted on the portal vein.
- ✓ During the perfusion it is possible to take samples of the solution through the needle-free valves (4,9)
- ✓ FOLLOW THE INSTRUCTIONS DESCRIBED IN THE VITASMART USER MANUAL.